EN - Profin® Lamineer® IPR (InterProximal Reduction)

Tips help manage interproximal reduction safely and esthetically. Providing maximum control and efficiency, the 1.2mm reciprocating motion prevents damage to gingival tissue. A simple swap of tips allows for versatility to contour and reshape embrasures to their natural esthetic.

INSTRUCTIONS FOR USE

- The Lamineer® IPR Tips are to be used with the Profin contra-angle.

- Determine and review the prescribed amount of IPR. IPR should be determined by final osition of the teeth. In cases of severe crowding, IPR can be staged later in treatment in order to await better access to interproximal contacts.

- Insert the Lamineer Tip into the Profin contra-angle making sure the tip is locked in a fixed position.

- The Profin contra-angle should be operated at 8000-10000 RPMs. Start the contra-angle prior to contact with the tooth surface.

- The Lamineer Tip should be used with a gentle, slow but continual movement through the contact. Coolant is not required due to the short reciprocating movement (1.2 mm).

- Confirm amount of removal using the Lamineer IPR Tip as your gauge (refer to color coded chart for sizes).

It is recommended that the Lamineer Tungsten polishing tips are used for subsequent polishing of the treated tooth surfaces to minimize scratching on the enamel. Fluoride treatment is often advised to strengthen the treated surface.

Maintenance: Remove debris clogging by running the diamond coated side against a cleaning block for diamonds. Clean the tips in an ultrasonic cleaner. Sterilize the tips in an autoclave maximum 134°C.

All Lamineer Tips are intended for re-use and should be thoroughly cleaned after each use.

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