
Elypse™ platforms

EN - Instruction for use
DE - Gebrauchsanweisung
IT - Istruzioni per l'uso
FR - Mode d'emploi
ES - Instrucciones de uso
SE - Bruksanvisning
NL - Instructions
LT - Instrukcijos
EL - Οδηγίες χρήσης
PL - Instrukcja użytkowania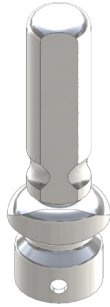

CE
0413

Catalog no
REF / Item


Product Description

ET-K	3x Denture Forming Instruments
	4x ANEW Elypse Platforms, Ø 3,5 mm
	4x ANEW Elypse Platforms, Ø 4,0 mm
	10x Marking Caps
	1x ANEW Elyps Dismantling Driver
	1x Instruction for use
ET-190	4x ANEW Elypse Platforms, Ø 3,5 mm
ET-192	4x ANEW Elypse Platforms, Ø 4,0 mm

ANew[®] Implants

		Diam (mm)	Tot. length (mm)	Thread length (mm)
ST-18S-R	ANEW Implant Pack	1.8	14	7
ST-18M-R	ANEW Implant Pack	"	17	10
ST-18L-R	ANEW Implant Pack	"	21	14
ST-22S-R	ANEW Implant Pack	2.2	14	7
ST-22M-R	ANEW Implant Pack	"	17	10
ST-22L-R	ANEW Implant Pack	"	21	14
ST-24S-R	ANEW Implant Pack	2.4	14	7
ST-24M-R	ANEW Implant Pack	"	17	10
ST-24L-R	ANEW Implant Pack	"	21	14
ST-28M-R	ANEW Implant Pack	2.8	17	10
ST-28L-R	ANEW Implant Pack	"	21	14

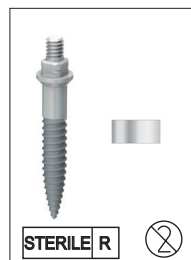ENGLISH	Instruction for use	5
DEUTSCH	Gebrauchsanweisung	11
ITALIANO	Istruzioni per l'uso	17
FRANÇAIS	Mode d'emploi	23
ESPAÑOL	Instrucciones de uso	29
SVENSKA	Bruksanvisning	35
NEDERLANDS	Instructions	41
LIETUVIŲ K.	Instrukcijos	47
ΕΛΛΗΝΙΚΑ	Οδηγίες χρήσης	53
POLSKI	Instrukcja użytkowania	59

Elypse™ Platforms for Anew Implants.

The Elypse Platforms offer an added feature to the Dentatus Anew Implant system. They provide the possibility to use overdentures during a transitional or long-term period and can later be removed and exchanged for a long-term fixed screw retained restoration.

Note: to facilitate retrievability to a fixed screw retained restoration, it is strongly recommended to use semi-permanent or temporary cement. Permanent cement may obstruct the removal of the Elypse Platform.

Instruction for Use.

The Elypse Platforms are intended to be used for improved retention of overdentures in the lower jaw during a transitional or long-term period, decided by the dental care provider. The platforms shall be used together with Anew implants and a soft silicone denture reliner, ie. Dentatus Tuf-Link^{II}. When an existing denture cannot be stabilized on the ridge aligned with the opposing teeth, the denture should be first relined with quick auto-cure resin before initiating the retrofitting procedure.

PRE-OPERATIVE PLANNING PROCEDURES PLACEMENT OF ANEW IMPLANTS AND ELYPSE PLATFORMS

- a.** Four Anew implants are placed between the two mental foramen in the mandible. The placement of the implants follows the procedure for Anew implants (see Anew Instructions for Use booklet).

- b.** Check that the upper level of the implant platform is flush with the gingival tissue.

- c.** Choose the appropriate size of Elypse Platforms. The S size (small) creates less retentive force and is recommended initially after surgery, during the osseointegration period. If a stronger retention is required at a later time, the larger Elypse Platform (L) can be used at that time.

- d.** Clean and dry the threads on the implants. Put a small amount of semi- permanent or temporary cement in the threaded opening of the Elypse Platform and on the thread of the implant. Use the built-on extension

handle to place the Elypse platforms on the implants. Tighten them by using finger power only and turn back until the longer side of the platform follows the curve of the jaw to minimize the bucco-lingual width. Let set. (Setting time of cement according to manufactures instructions)

- e.** When the cement has set, break off the handles from the Elypse platforms. Remove excess cement and the Gingival Spacers and clean the sulcus around each implant. If necessary, smoothen off the breaking point on top of the Elypse platform with a polishing diamond.


- f.** Put an Atlas Marking Cap on each Elypse platform. They are used for marking the position of the implants in the denture in the next step.

CLINICAL PROSTHETIC PROCEDURES

When an existing denture cannot be located spatially on the ridge aligned with the opposing teeth, the denture should be first relined with quick auto-cure resin before initiating the retrofitting procedure.

A marking pen is used to mark the outline in the denture for a seam line position that should be located 2-3 mm below the flange extensions, 1mm undercut, 4mm clearance from implants. The marking is extended up to the retromolar pads in a curved form for seamless liner adaptation.


1. The half-sphere cutting instrument is used to create an inverted seam line encasement for retention of the liner without adhesives. The instrument shank is leaned toward the opposing flange, angled at 45° to create a well delineated, continuous seam line.

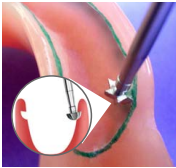
2. The grooved instrument is used to score depth tracks in the denture without causing damage to the prepared seam line.


3. The spherical cutting instrument is used to reduce the base to the depth marked levels and to create a smooth surface.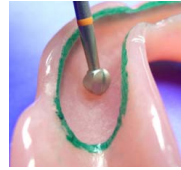

4. The marking caps, placed on the Atlas Implants are used to mark their position in the denture. The caps are painted with any safe vegetable-based colorant or marker and the denture is firmly pressed down to transfer the marked positions for the implant head.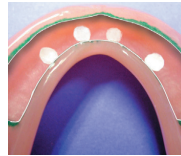

5. The spherical cutting instrument is used to indent the marked spaces allowing the denture to seat passively in place. To prevent excessive denture reduction, the instrument should be sunk down 2/3 of its spherical depth at the places of the Implants. When the marker paint no longer is transferred from the marking caps to the denture base, the reduction is sufficient.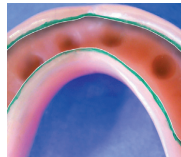

Note: A confirming check can be performed by applying markings directly to the implant heads for a final touch up and rounding off all sharp projections to prevent incipient tears in the silicone liner.

SILICONE RELINING PROCEDURES


Note: The patient's mouth, tongue and denture are thoroughly rinsed to remove fine particles that may become embedded in the liner. Preparing for the quick reline procedure, the patient should be cautioned that they will need to gently press the denture in place in contact with their opposing teeth and keep firmly in place for 4-6 minutes without tongue or jaw movements.

1. The Tuf-Link dispenser with an auto-mixing tip is bled before injecting the silicone into the denture base, the flow is guided into the seam line. Sufficient silicone should be used to create pressure required for a dense, nonporous liner.

2. The denture is inserted from the posterior direction and guided forward with pulsating side motions. The patient is guided to contact the opposing teeth and hold still in place. The procedure must be carried out expeditiously, as the silicone begins to set within minutes.
3. When polymerized, the denture is removed and the liner is gently lifted out of the denture base with a pointed instrument at one of the distal extensions.
4. Check that the silicone thickness is at minimum 2 mm above the implant heads. If not, adjust the denture base accordingly.

5. The excess liner is cut away along the well delineated seam line with long-nosed fine scissors.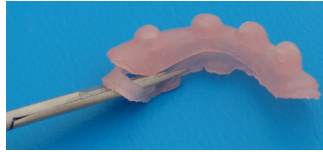

6. To reinsert the liner, it is first located at one extreme end of the denture and guided into place with light finger pressure. A smooth flat instrument may be used to guide the reline into the seam line which will prevent fine food particle accumulation.

Atlas Tuf-Link self-aligning dentures tolerate substantial implant angulations. The denture, removed without implant stress or tissue abrasions, rarely requires follow-up tissue adjustment appointments. Depending on variable conditions, the liners should remain resilient for 12 - 18 months. Although they are quickly replaceable, a spare liner can be made for patients to keep it in a water filled container for emergency use.

POST-OP CHECKLIST

- a.** Muscle impinging areas, the frequent cause of denture dislocation in functional and parafunctional movements should be eliminated.
- b.** Occlusal contacts and tripping interferences should be corrected.
- c.** The patient should immediately report any irregularities or discomfort.

PATIENT HOME-CARE INSTRUCTIONS

Patients can immediately ingest food; however, they should observe 24 - 48 hour soft diet regimen, without hard or chewy foods.

- a.** Patient should not remove their denture during the first two days.
- b.** Patient should not remove their denture at night for the first two weeks following Atlas placement. This will prevent any edema initiated by the surgical placement from forcing the soft tissue up, possibly interfering with the reseating of the denture on the implants. After the two week period patients may sleep with or without their dentures.

- c.** Patient should remove denture briefly each day for oral hygiene. Instruct them to use a soft nylon bristle toothbrush with toothpaste or water to scrub the denture. Do not have patient attempt to remove the silicone liner for at least 3 days. After this initial period, patients with good dexterity may remove the liner (with a toothpick) and clean it gently in the same manner as the denture. After the initial healing period, dentures may be soaked for 30 minutes each day in a denture cleaning solution.

(Note: instruct patients not to clean their dentures or silicone relines over an open sink drain). Instruct patients how to clean the head, collar and platform of the Atlas implants.

- d.** Patient may use a pain-relieving medication to alleviate any discomfort.
- e.** Patient should be instructed to call the office if any serious pain or other symptoms arise.

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