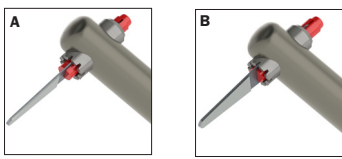
Multilingual A: EN, SV, DE, ES, FR, IT, BG, CS, DK, EL, FI

Profin PDH

EN - Profin PDH Instruction for use – maintenance information

Prior to initial use - always read the instructions for use. Only for USA: prescription use – for use by dental clinicians only.

Indication for use
Finish and polish tooth surfaces.

Precaution
Insert the shank of the tip into the holder. Check that the tip fits safely by pulling the tip gently with your fingers. A firm resistance should be sensed.

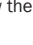
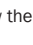
Caution
If a tip comes out too easily from the holder, discard and use another tip.

| Technical description | Recommendation of use |
|---|--|
| Holder | |
| Profin PDH angled end | For use in distal areas |
| Profin PDH straight end | For use in anterior areas |
| Variation of mode | |
| A. Tip in a self-steering mode (The tip can rotate along its axis.) | For following the contour of the tooth easily. Normally used during polishing and finishing. |
| B. Tip in fixed mode in one of the indexed positions. Eight positions/settings. | For improved tactility/accuracy. E.g. for creating individual structures for aesthetic appearance. - To change the position - pull out the tip slightly, turn into desired position and push it back into a suitable notch in the holder. |
| Durability of tips | |
| Diamond or Tungsten Carbide coated | The durability depends on the circumstances, for continued efficiency it is recommended not to use them more than 30-40 times. |

Recommendations

- A general recommendation is to use a more coarse tip when larger amount of material is to be removed. Removal of material should always be made successively from tips with more coarse grit to finer.
- Do not change directly from super-coarse to super-fine tips in one step, but use intermediate in coarse-m edium.
- The black Tungsten-Carbide-coated tips can always be used for finishing when there is a risk for scarring enamel, since Tungsten-Carbide does not scar or suff enamel.

Maintenance Profin PDH

| | | |
|---|--|--|
| Limitations of reprocessing | Check for wear of holders | |
| 1. Preparation at the point of use | None. | |
| 2. Decontamination | | |
| Cleaning | Manual Flush off larger debris under running water. Use a not-metal brush to avoid scratching the instrument. Wipe off with a soft cloth. | |
| |  | Without or with mild detergents. Always follow the manufacturer's instruction. |
| Disinfection |  Maximum 95 °C (203 °F). Without or with mild detergents. Always follow the manufacturer's instruction. | |
| Drying | Let air dry or wipe off with dry cloth. | |
| Inspection | Visually inspect for appearance and function. | |
| 3. Sterilization - optional | | |
| |  Place in sterilization bag of paper/clear film (according to applicable standards). Check correct sealing of the bag. 134 °C (273 °F) | |
| Storage | Protect from dust and dirt. | |

LTA-tips

| | |
|---|---|
| Limitations of reprocessing | See Durability above. |
| 1. Preparation at the point of use | Check for wear of tip |
| 2. Decontamination | |
| Cleaning | Without or with mild detergents. Always follow the manufacturer's instruction. |
| | Manual Rub against the cleaning stone to remove clogged debris, flush under running water. |
| Disinfection | Thermal disinfectors Maximum 95 °C (203 °F). Without or with mild detergents. Always follow the manufacturer's instruction. |
| Drying | Let air dry. |
| Inspection | Visually inspect for appearance and function. Eliminate worn tips. |
| 3. Sterilization - optional | |
| | Place in sterilization bag of paper/clear film (according to applicable standards). Check correct sealing of the bag. 134 °C (273 °F) |
| Storage | Protect from dust and dirt. |

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Dentatus AB