

DENTATUS ARTICULATORS

FOR EFFICIENT AND ACCURATE STUDY OF OCCLUSAL RELATIONS

Sturdiness and Precise Performance

The Advantages

- Robust, precise and reliable
- New and upgraded design for easier usage and maintenance
- Ergonomic knobs and improved visibility scales
- Wide range of accessories and spare parts
- Can be used with Dentatus' Face and Ear Bows

We are proud to introduce two upgraded Dentatus articulators, ARH2 and ARL2, in a new modern design for easier usage and maintenance.

The new upgraded models in the Dentatus articulator and face bow range keep the same high quality and precision as recognized by the profession for many years. Main improvements are found with scales and settings which have been made more ergonomical for the user.

Dedicated face & ear bows

Precise and reliable performance

Swedish quality



Dentatus


MADE IN
SWEDEN

DENTATUS ARTICULATORS

The design of the Dentatus articulators has been made to fulfill the requirements necessary for efficient and accurate study of occlusal relations, diagnosis and treatment planning in natural and artificial dentitions. It is a high quality tool for individual reproduction of tooth and jaw relations, for analysis and extra oral manufacturing of individual prosthetics and other individual related dental devices.

The Dentatus articulators are made of aluminium and high quality stainless steel with anodized surfaces (black details) and powder coated (blue details) for sturdy and reliable use over long time. Durable plastic feet protect table surfaces from scratches.

The care with which polished or anodized surfaces, ground parts, etc. have been machined and finished is not only intended for the appearance but also to facilitate the removal of plaster, wax and other foreign matter from the articulator.

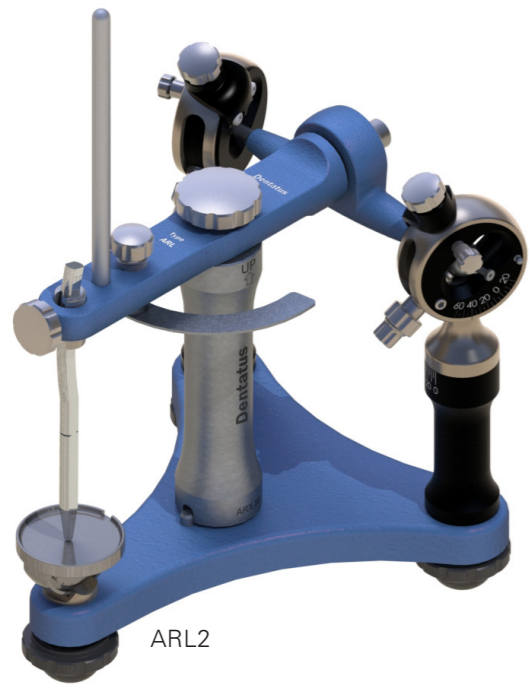

ARL2



ARL2 without Gauge Block

Dentatus Articulator ARL2 – Top Of The Line

Specifications:

- Adjustable Upper Jaw Member - Easily calibrated by use of the Gauge Block.
- With the Gauge Block the articulator can be adjusted and will assure correct basic settings when cooperating dentists and laboratories work with their individual ARL2 articulators.
- An articulator will in practical work be subject to gradual changes either by wear or by accident. When such changes are discovered, checking and adjustment of the articulator is important. The Gauge Block is an efficient adjunct to the ARL2 Articulator for continued precision over time.
- Adjustable Condylar Track Inclination -20° to $+60^{\circ}$
- Full range Bennett shift 0° to 40°
- The Condylar Mechanism can be set from individual records
- Fixed intercondylar distance: 95mm
- Opens $180^{\circ}+$
- Condylar Track Assemblies equipped with Special Auditory Pins for attachment of the Dentatus Ear Bow AEE
- Improved thread design to minimize unintended lock-in by stone
- Solid incisal table, can be tilted anterior-posteriorly, and detachable – can be interchanged with the Fully Adjustable Segmented Incisal Table
- Total model height: 105mm
- Corrosion resistant
- Weight: 1038 g

MADE IN SWEDEN SINCE 1944

Dentatus Articulator ARH2 – A True Companion

Specifications:


As Dentatus ARL2 articulator but without the Adjustable Upper Jaw Member and without calibration possibility with the Gauge Block

- Adjustable immediate side shift
- Weight: 1010 g




ARH2

The Articulator

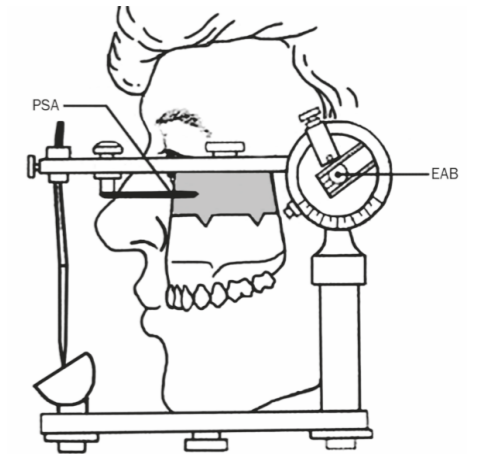

The articulator is defined as "a mechanical instrument that represents the temporomandibular joints (TMJs) and jaws, to which maxillary and mandibular casts may be attached to simulate some or all mandibular movements" (Academy of Denture Prosthetics 1977).

The Importance of Occlusion

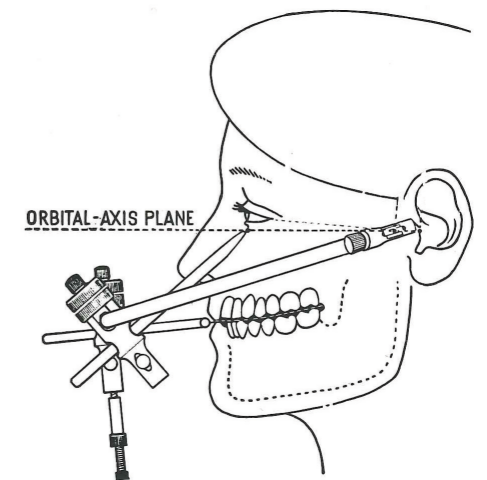

Understanding the correct relations between the static and functional dental contacts is key to good clinical practice in restorative dentistry. The articulator's primary function is to work as if it was a patient 'in-absence'.


Made in Sweden Since 1944

The Dentatus articulators have been manufactured in Sweden since 1944 and are still well-known high-quality precision instruments designed for long time service at dental clinics, laboratories and education centers across the world. For more information, please visit Dentatus' website www.dentatus.com


Service and Spare Parts

We give service and exchange worn parts on several of our old models. Contact our Service Department for service and spare parts via email, info@dentatus.se.

USE DENTATUS' FACE BOW AND/OR EAR BOW WITH YOUR DENTATUS ARTICULATOR

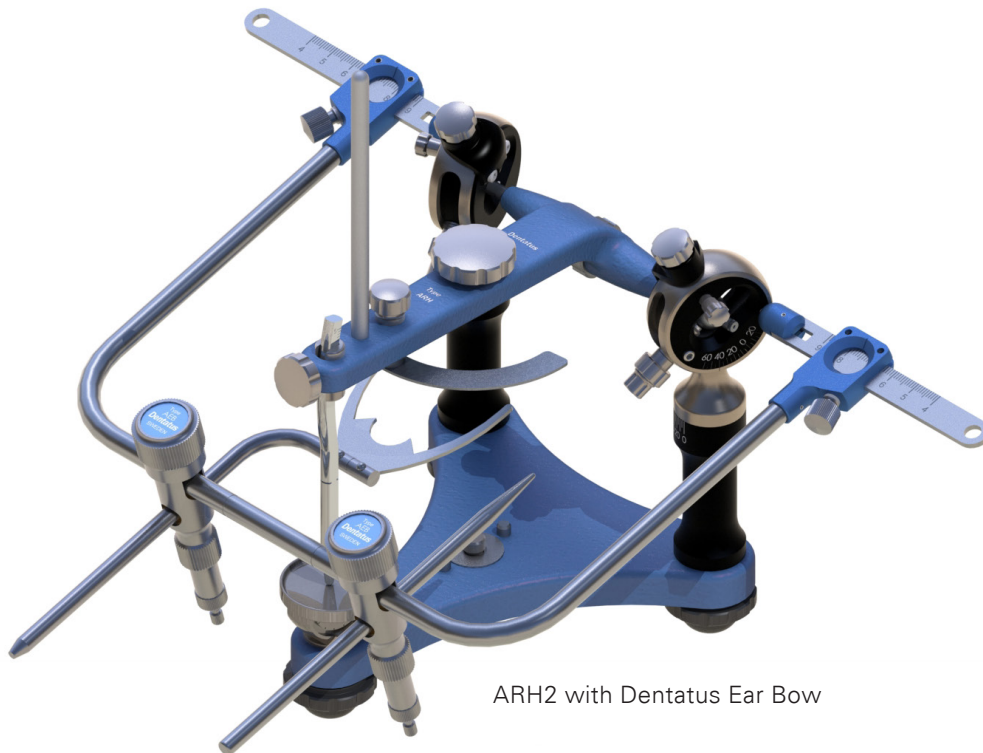

ARH2 with Dentatus Ear Bow

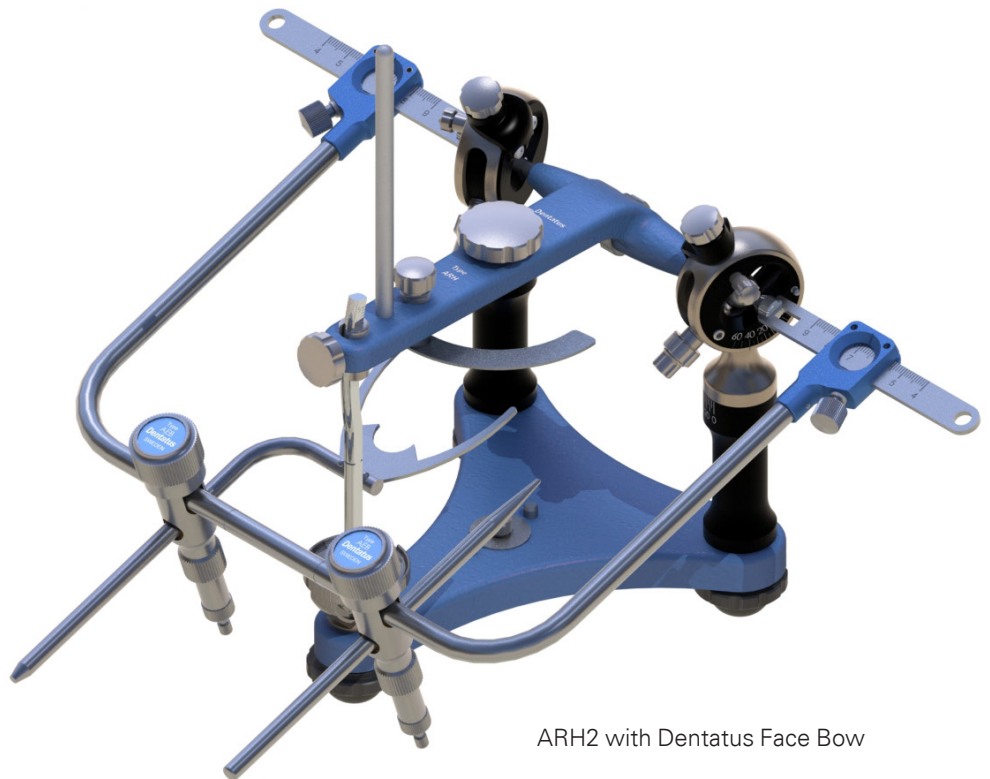

ARH2 with Dentatus Face Bow



Dentatus AB
www.dentatus.com
SE: +46 8 546 509 00
info@dentatus.se
US: +1 212 481 1010
dentatus@dentatus.com

