
## PLANAS ACCESSORY SET FOR ARL2

## FACE BOWS & EAR BOWS, AEE, AEB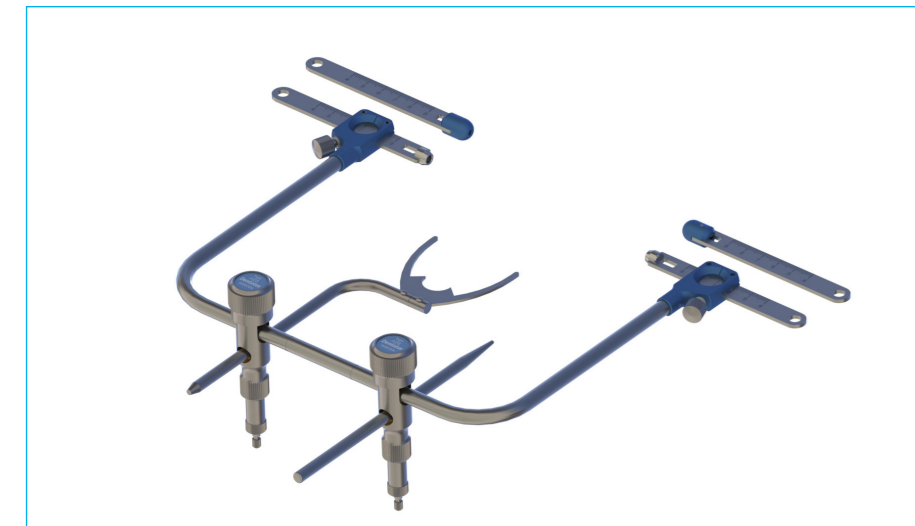

**EN:** Use medical gloves when handling the device.  
**DE:** Verwenden Sie beim Umgang mit dem Gerät medizinische Handschuhe.  
**EL:** Χρησιμοποιήστε ιατρικά γάντια όταν χειρίζεστε τη συσκευή.  
**ES:** Utilice guantes médicos al manipular el dispositivo.  
**FI:** Käytä lääketieteellisiä käsineitä laitetta käsitellessäsi.  
**FR:** Utilisez des gants médicaux lors de la manipulation de l'appareil.  
**IT:** Utilizzare guanti medici durante la manipolazione del dispositivo.  
**LT:** Dirbdami su prietaisu naudokite medicinines pirštines.  
**SV:** Använd medicinska handskar när du hanterar enheten.

**CE** **EN:** CE marking / **DE:** CE-Kennzeichnung / **EL:** Σήμανση CE / **ES:** Marcado CE / **FI:** CE-merkintä / **FR:** Marquage CE / **IT:** Marcatura CE / **LT:** CE ženkinimas / **SV:** CE märkning

**EN:** Manufacturer / **DE:** Hersteller / **EL:** Κατασκευαστής / **ES:** Fabricante / **FI:** Valmistaja / **FR:** Fabricant / **IT:** Fabbrikante / **LT:** Gamintojas / **SV:** Producent

**REF** **EN:** Catalogue number / **DE:** Katalognummer / **EL:** Αριθμός καταλόγου / **ES:** Numero de catalogo / **FI:** Luettelon numero / **FR:** Numéro de catalogue / **IT:** Numero di catalogo / **LT:** Katalogo numeris / **SV:** Katalognummer

**SN** **EN:** Serial number / **DE:** Seriennummer / **EL:** Σειριακός αριθμός / **ES:** Número de serie / **FI:** Sarjanumero / **FR:** Numéro de série / **IT:** Numero di serie / **LT:** Serijos numeris / **SV:** Seriennummer

**UDI** **EN:** Unique Device Identification / **DE:** Eindeutige Geräteidentifikation / **EL:** Μοναδική αναγνώριση συσκευής / **ES:** Identificación única de dispositivo / **FI:** Yksilöllinen laitetunniste / **FR:** Identification unique de l'appareil / **IT:** identificativo unico del dispositivo / **LT:** Unikalus įrenginio identifikavimas / **SV:** Unik enhetsidentifisering

**EN:** Date of manufacture / **DE:** Herstellungsdatum / **EL:** Ημερομηνία παραγωγής / **ES:** Fecha de fabricación / **FI:** Valmistuspäivämäärä / **FR:** Date de fabrication / **IT:** Data di produzione / **LT:** Pagaminimo data / **SV:** Tillverkningsdatum

**MD** **EN:** Medical Device / **DE:** Medizinprodukt / **EL:** Ιατρική συσκευή / **ES:** Dispositivo médico / **FI:** Lääkinnällinen laite / **FR:** Dispositif médical / **IT:** Dispositivo medico / **LT:** Medicininis prietaisas / **SV:** Medicinsk utrustning

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











BL-6756-01\_2021-09-13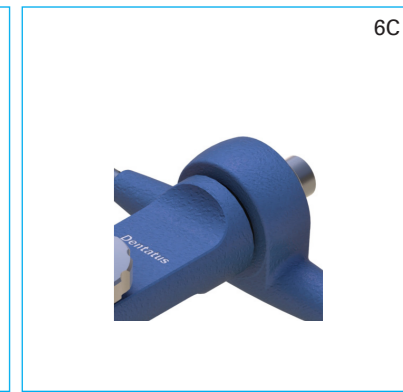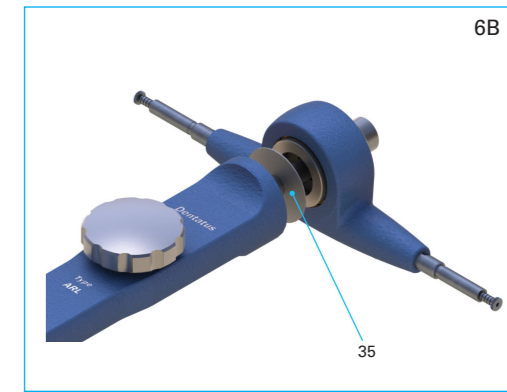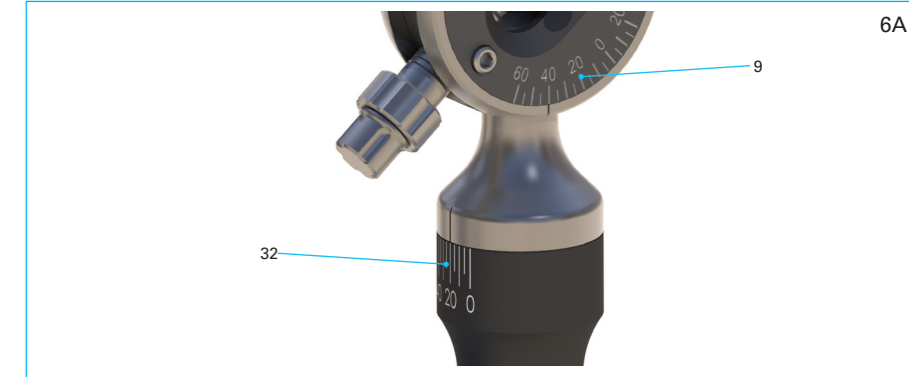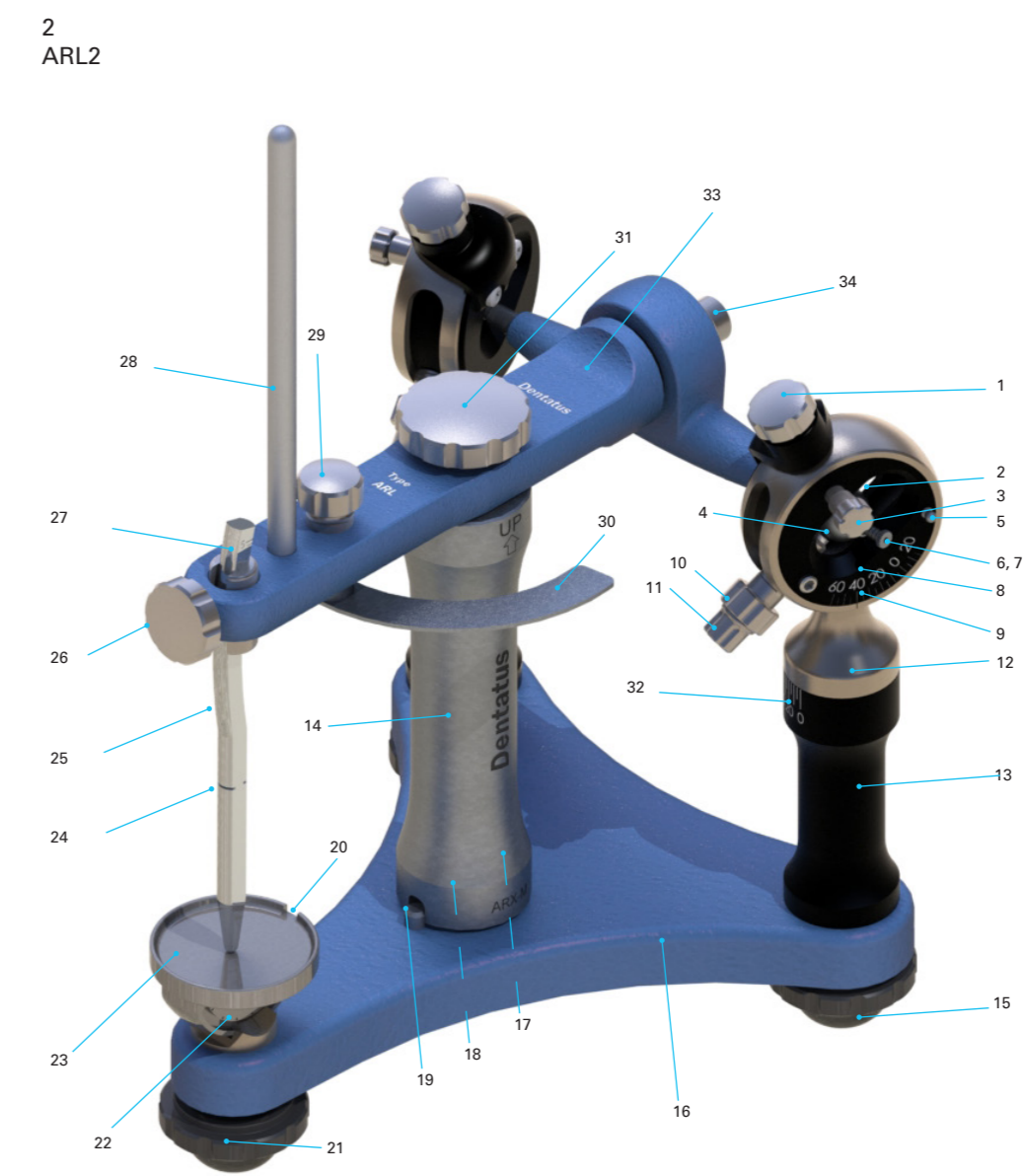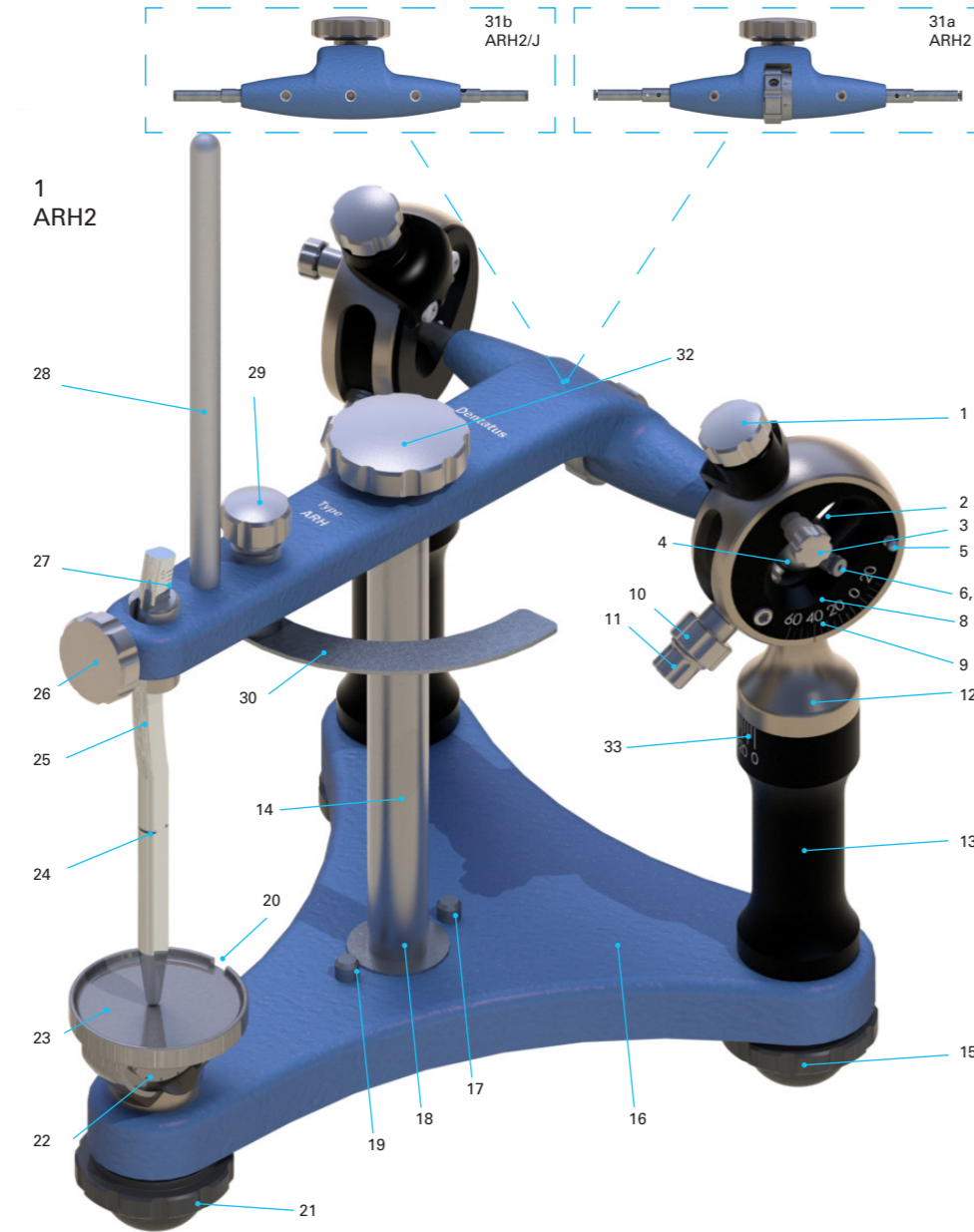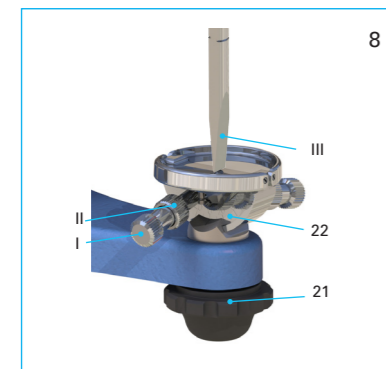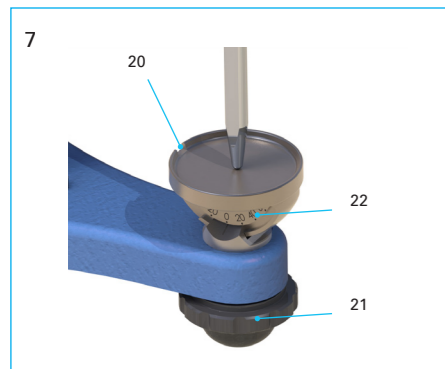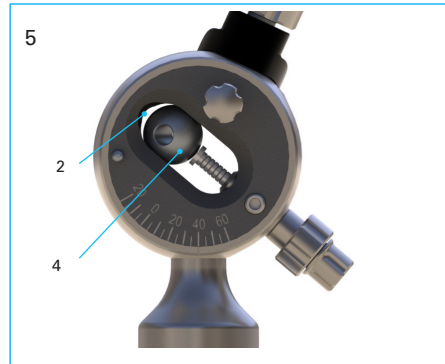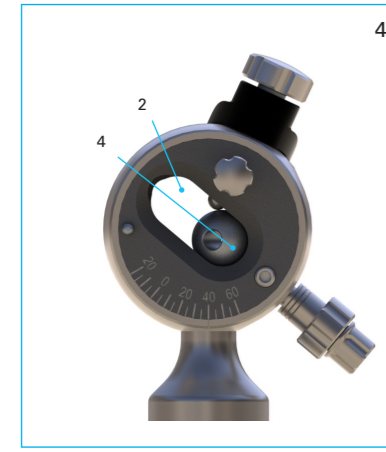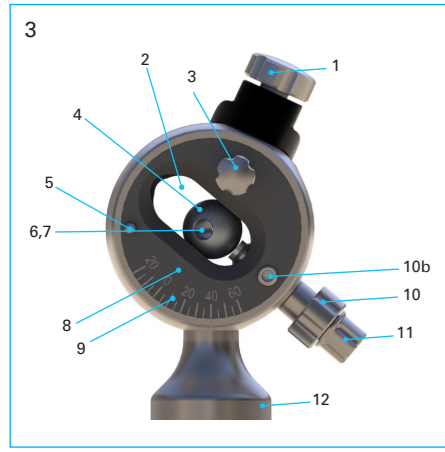

MOUNTING PLATES, ARH-Z, ARH-ZC
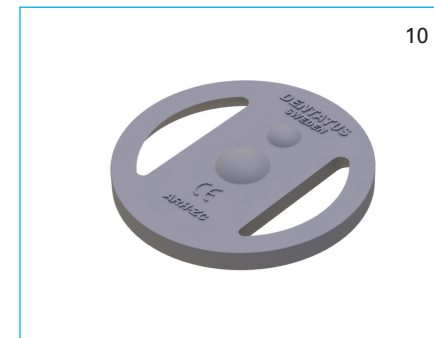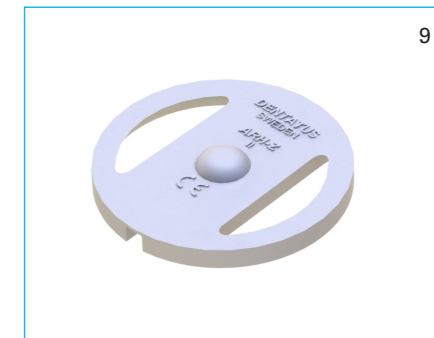
## **We are glad to see that you have chosen one of Dentatus' high quality Articulators, made in Sweden.**

### Introduction

The Dentatus articulators are designed to fulfil high expectations in usability over a long period of time in demanding dental environments.

You will find all illustrations on the cover of this booklet.

### Intended use

It is intended as a tool for individual reproduction of tooth and jaw relations for analysis and extraoral manufacturing of individual prosthetics and other individual related dental devices.

### Risks

No risks have been revealed for patients or users at dental clinics and laboratories.

### Maintenance

This articulator is a precision instrument and must be treated as such. Care, diligence and cleanliness should be used in its handling. This will help to secure the proper accuracy of the articulator.

Wipe off or remove gypsum/stone materials as well as molten wax immediately. Material residues in any moving part or surface may reduce correct function and obstruct proper use.

**Do not use corrosive cleaning agents, as this may affect the metal parts of the articulator.**

Protect the articulator from bumps and blows.

### Materials

Stainless steel, brass, aluminum, thermoplastic polymers.

### Service

Although the Dentatus articulators are sturdy and designed for long time service at dental clinics and laboratories, components may get lost or accidentally damaged.

For replacement parts or service – contact your local dealer.

## Dentatus articulators ARH2 & ARH2/J, descriptions

*see illustrations on cover, fig 1 (left)*

1. Set Screw for Horizontal Condylar Inclination (HCI)
2. Condylar Track
3. Lock Screw for Condylar Sphere
4. Condylar Sphere
5. Auditory Pin
6. Condylar Axis with Axis Pin
7. Centerpunched Calibrated Condylar Axis Pin
8. Condylar Track Assembly
9. HCI Calibration Scale
10. Larger Anterior Stop Screw for Condylar Sphere
11. Smaller Anterior Stop Screw for Condylar Sphere
12. Condylar Assembly Holder
13. Condylar Post
14. Transport Safety Post (Locking Column)\*
15. Lock Nut for Bennett-Angle Setting
16. Lower Jaw Member
17. Posterior Positioning Pin for Mounting Plate
18. Attachment Screw for Lower Mounting Plate
19. Anterior Positioning Pin for Mounting Plate
20. Incisal Table Groove for Positioning of Individual Anterior Guidance
21. Lock Screw for Incisal Table angulation
22. Calibration for Incisal Table (Setting is engraved on opposite side)
23. Solid Detachable Incisal Table
24. Incisal Groove
25. Calibrated Curved Incisal Pin
26. Lock Screw for Incisal Pin
27. Incisal Pin Calibration Scale
28. Support Rod for Upper Jaw Member
29. Lock Nut for Orbital Axis Plane Indicator
30. Orbital Axis Plane Indicator
31. Sideshift setting thumbwheel
- 31a. ARH2 with Sideshift
- 31b. ARH2/J without Sideshift
32. Attachment Screw for Upper Mounting Plate
33. Bennett-Angle Calibration Scale

\* The locking column (14) with ARH2 and ARH2/J is a protective device and should be in place during transport. Casts should be detached and wrapped separately during transport.

## Dentatus articulator ARL2, descriptions

*see illustrations on cover, fig 2 (right)*

1. Set Screw for Horizontal Condylar Inclination (HCI)
2. Condylar Track
3. Lock Screw for Condylar Sphere
4. Condylar Sphere
5. Auditory Pin
6. Condylar Axis with Axis Pin
7. Centerpunched Calibrated Condylar Axis Pin
8. Condylar Track Assembly
9. HCI Calibration Scale
10. Larger Anterior Stop Screw for Condylar Sphere
11. Smaller Anterior Stop Screw for Condylar Sphere
12. Condylar Assembly Holder
13. Condylar Post
14. Gauge Block\*
15. Lock Nut for Bennett-Angle Setting
16. Lower Jaw Member
17. Posterior Positioning Pin for Mounting Plate
18. Attachment Screw for Lower Mounting Plate
19. Anterior Positioning Pin for Mounting Plate
20. Incisal Table Groove for Positioning of Individual Anterior Guidance
21. Lock Screw for Incisal Table angulation
22. Calibration for Incisal Table (Setting is engraved on opposite side)
23. Solid Detachable Incisal Table
24. Incisal Groove
25. Calibrated Curved Incisal Pin
26. Lock Screw for Incisal Pin
27. Incisal Pin Calibration Scale
28. Support Rod for Upper Jaw Member
29. Lock Nut for Orbital Axis Plane Indicator
30. Orbital Axis Plane Indicator
31. Attachment Screw for Upper Mounting Plate
32. Bennett-Angle Calibration Scale
33. Adjustable Upper Jaw Member
34. Universal Joint Lock Screw

\* The Gauge Block (14) with ARL2 is also a protective device and should be in place during transport. Casts should be detached and wrapped separately during transport.

## GENERAL FEATURES - STANDARD SETTINGS

*see illustrations on cover, fig 3-5 (left)*

1. Set Screw for Horizontal Condylar Inclination (HCI)
2. Condylar Track
3. Lock Screw for Condylar Sphere
4. Condylar Sphere
5. Auditory Pin
6. Condylar Axis with Axis Pin
7. Centerpunched Calibrated Condylar Axis Pin
8. Condylar Track Assembly
9. HCI Calibration scale
10. Larger Anterior Stop Screw for Condylar Sphere
- 10b. Lock Screw for Larger Anterior Stop Screw for Condylar Sphere.
11. Smaller Anterior Stop Screw for Condylar Sphere
12. Condylar Assembly Holder

The Condylar Sphere (4) is in its BASIC POSITION when both the Larger Anterior Stop Screw (10) and the Smaller Stop Screw (11) for Condylar Sphere are turned as far as possible in a clockwise direction.

The Smaller Anterior Stop Screw (11) for Condylar Sphere has a left hand thread: It will move into the Condylar Track (2) when it is turned counterclockwise.

Note: Condylar Spheres MUST always be FULLY UNLOCKED during any adjustments of the Anterior Stop Screws (Lock Screws for Condylar Spheres (3) should be turned all the way counter clockwise)

The Condylar Spheres (4) should both be placed and locked in their BASIC POSITION during routine work with the Articulator.

These adjustments enhance the reproduction of individual jaw movements.

*Figure 3, Condylar Sphere in Basic Position in Condylar Track.*

<p><u>Standard settings</u> for HCI: 40° for Bennett-Angle: 20°</p>
---

*Figure 4, Condylar Sphere (4) moved Anteriorly in Condylar Track (2) imitates a retrusive movement of the mandible.*

*Figure 5, Condylar Sphere (4) moved posteriorly in Condylar Track (2) imitates a protrusive movement of the mandible.*

## TO CALIBRATE DENTATUS ARTICULATOR ARL2

*see illustrations on cover, fig 2, 6 (right)*

1. Set and lock calibrations for HCI (9) at 40° and Bennett-Angle Calibrations (33) at 20°. - Lock Condylar Spheres (4) with their Lock Screws (3) in Articulator's Basic Position. - See fig 6A. Raise the Incisal Pin (25).
2. Clean meticulously the bushings on the Upper and Lower Jaw Members (16, 33), against which the Gauge Block will be placed.
3. Mount the Gauge Block (31).
4. Insert a 5 mm Allen Key in Universal Joint Lock Screw (34) and loosen it, turning the key counter-clockwise.
5. Check if lock washer (fig 6B, 35) is present between the Universal Joint Connection and the Upper Jaw Member.
6. Check that space between the two Upper Jaw Member Parts (fig 6C) is the same all around.
7. Hold the two upper member parts in the prescribed position and tighten Universal Joint Lock Screw (34) lightly with key.
8. After adjustment - Loosen Condylar Lock Screws (3). Check that the Condylar Spheres remain in their basic position. Otherwise repeat adjustment.
9. Remove Gauge Block (14)
10. Relock Condylar Lock Screws (3) and tighten Universal Lock Screw solidly.  
**IMPORTANT: THIS TIGHTENING MUST BE MADE FORCEFULLY.**
11. The Articulator is now properly adjusted and ready for use.

## DENTATUS ARTICULATORS ACCESSORIES, *see illustrations on cover, fig 1, 7, 8 (left)*

### Solid Detachable Incisal Table

The Solid Incisal Table (23) can be tilted anterior-posteriorly. The Table is detachable and can be interchanged with the fully adjustable Segmented Incisal Table shown in figure 8.

*Figure 7*

*20. Incisal Table Groove for Positioning of Individual Anterior Guidance*

*21. Lock Screw for Incisal Table*

*22. Calibration for Incisal Table Setting*

### Interchangeable segmented incisal table

The Segmented Incisal Table can be adjusted and locked in the desired position. It requires a Special Incisal Pin with a flat tip (III), which allows immediate rise in any lateral movement.

The Incisal Table can be tilted anterior-posteriorly in the horizontal plane and locked in the desired position by its Lock Screw (21).

*Figure 8*

*I Special Set Screw*

*II Lock Nut for Set Screw*

*III Special Incisal Pin with broad tip*

*21. Lock Screw for Incisal Table*

*22. Calibration for Incisal Table Setting*

### Mounting plates

*fig 9-10 (right)*

The Mounting Plate ARH-Z is cast of zinc. The ARH-ZC is entirely molded in fiber reinforced composite. Lighter in weight and easier to clean, but with an equally high stability as the zinc plates.

*Figure 9*

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