
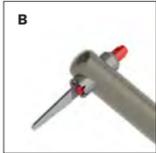
Profin[®] PDH

EN, BG, CS, DA, DE, EL, ES, FI, FR, IT, LT, NO, NL, PL, PT, RO, SK, SV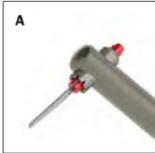

CE Rx Only

BL-6734-01_2021-09-24

EN - Profin PDH Instruction for use – Maintenance information

Prior to initial use - always read the instructions for use.

				Sterilizable in a steam sterilizer (autoclave) at the specified temperature
CE marking	Catalogue number	Lot number	Ultrasonic bath	
Medical Device	Unique Device Identification	Date of manufacture	Manufacturer	

Intended Use:
Handle for holding LTA and EVA tips.
Indication for Use:
To be used for manual shaping, finishing and polishing of teeth and dental restorations in connection with dental treatment.
Precaution/Warnings
Small objects: Prior to using, check for secure seating of tip in handle by pulling the tip with your fingers. A firm resistance should be sensed. If tip comes out easily, try another tip or replace handle.
Cross-contamination: Clean, disinfect and sterilize before first use and between uses.
The handle must be cleaned and disinfected before sterilization. Steam sterilizer shall be in accordance with standards EN 13060, EN 285 and EN ISO 17665-1. See Lamineer Tip IFU for decontamination and sterilization instructions of Lamineer Tips.

Technical Description	Recommendation of Use
Holder	
Profin PDH angled end	For use in posterior areas
Profin PDH straight end	For use in anterior areas
Variation of mode	
A. Self-steering mode (The tip rotates along its axis.)	Follows the contour of the tooth easily. Used during polishing and finishing.
B. Fixed mode, eight indexed position options.	Improves tactility/accuracy, e.g., for creating individual structures for aesthetic appearance. - To change the position – pull out the tip slightly, turn into desired position and push it back into a suitable notch in the holder.

Maintenance, Profin PDH	
WARNINGS	Only use chemicals according to manufacturer’s instruction and suitable for anodized aluminium.
Limitations of reprocessing	Check for wear of holders after each cleaning cycle. Do not use washer disinfector. Anodized aluminium is not suitable for washer disinfector. The instrument is not tested for repeated cycles in washer disinfector.
1. Preparation before cleaning	Ensure the tip is removed from the handle. If heavily soiled, rinse and brush under running lukewarm water until visibly clean. The maximum period of time between start of use and start of cleaning process shall not exceed 2 hours. ¹⁾
2. Decontamination	
Cleaning: Manual	External Equipment: Non-abrasive, medium hard brush. Detergent: Low alkaline Water quality: Deionized (DI) <40°C or Reverse osmosis (RO) <40°C Instructions: - Brush off under non-stagnant water. - Rinse under non-stagnant water for 15 seconds. Internal Equipment: Cleaning spray/dry spray intended for dental handpieces. Instructions: In accordance with manufacturer’s specification for the use of the spray(s). Hold handle against clean, white wipe and repeat procedure until no visible soil remains.
Ultrasonic	Equipment: Ultrasonic bath. Detergent: Mild low alkaline detergent. Temperature: 60 °C maximum Instructions: Follow manufacturer’s instructions.
Disinfection: Manual	Caution: The handle must be cleaned both internally and externally before disinfection. External Equipment/Detergent: Soaked cloth or pre-soaked wipes/Disinfectants suitable for anodized aluminium. Instructions: In accordance with manufacturer’s specification for the use of the disinfectants. Internal Equipment: Disinfection spray intended for dental handpieces. Instructions: In accordance with manufacturer’s specification for the use of the disinfection spray.
Inspection and testing	Inspection: Visually inspect for damage and wear. Testing: Check the head by inserting a tip. You should not be able to easily pull out the tip with your finger. If this occurs, do not use the handle and contact the manufacturer.
Packaging for sterilization	Equipment: Sterile goods packing. ²⁾ Instructions: - Pack individually - Check that the bag is not stretched - Check correct sealing
Sterilization	Caution: The handle must be cleaned and disinfected before sterilization. Equipment: Saturated steam sterilizer including fractionated pre-vacuum before sterilization phase and final post-vacuum drying (B-process). ³⁾ Temperature: 134 °C (273 °F) Time: Minimum of 3 minutes Pressure: 3.06 bar (27 psi). Vacuum drying: Minimum 3 minutes.
Storage	Store at room temperature. Protect from dust, dirt, humidity and UV-light (sunlight). Before use, ensure the sterile bag is dry, whole and clean.
Additional information	¹⁾ If delayed cleaning, corrosion may affect the surface. ²⁾ Sterile goods packing according to EN 868-5. ³⁾ Steam sterilizer shall be in accordance with standards EN 13060, EN 285 and EN ISO 17665-1. All equipment, detergents, spray, lubricants and disinfections solutions shall be CE-marked.

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