
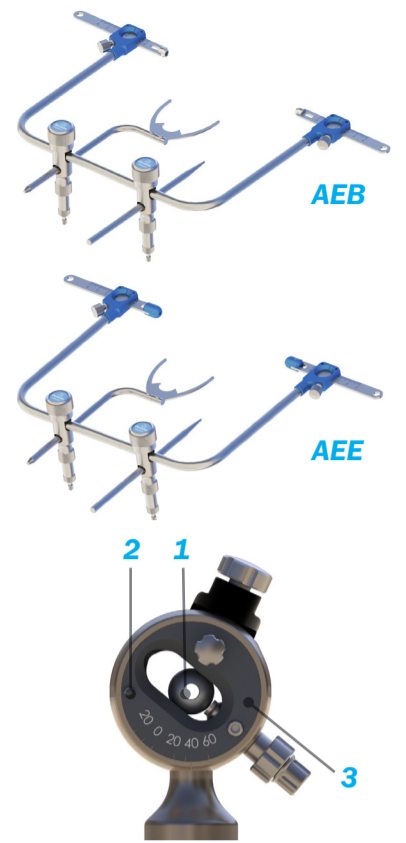
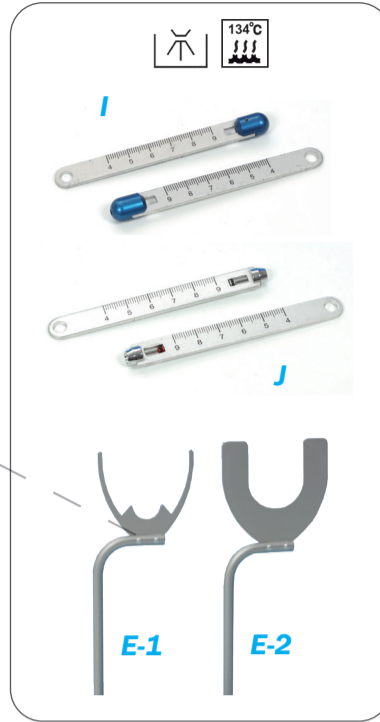
DENTATUS FACE BOWS TYPE AEB AND AEE

EN, DE, EL, ES, FI, FR, IT, LT, SV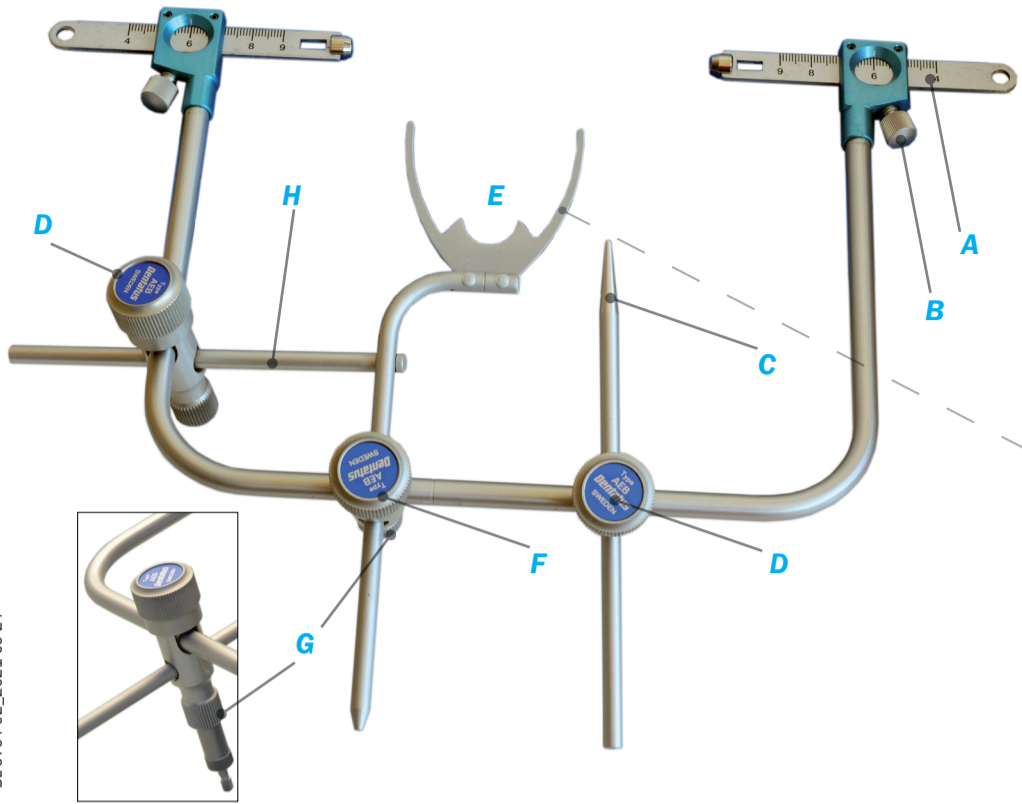

EN: Dentatus Articulator Face Bows, Specifications

					Washer-disinfector for thermal disinfection
CE marking	Catalogue number	Lot number	Caution		
					Sterilizable in a steam sterilizer (autoclave) at the specified temperature
Medical Device	Unique Device Identification	Date of manufacture	Manufacturer		

- A. Calibrated Condylar Rod (on **AEB**) - **AEB-B2**
- B. Set Screw for Calibrated Condylar Rod
- C. Orbital Pointer Pin - **AEB-18**
- D. Locking Clamp for Orbital Pointer Pin and for Support Rod - **AEB-R**
- E-1. Bite Fork, Standard - **AEB-G**
- E-2. Bite Fork, filled (optional) - **AEB-J**
- F. Locking Clamp for Bite Fork complete with Anterior Jack Screw - **AEB-Y**
- G. Anterior Jack Screw - **AEB-L**

- H. Support Rod for Bite Fork - **AEB-19**
 - I. Calibrated Condylar Rods with blue rounded earpieces (on **AEE**) - **AEE-E2**
 - J. Calibrated Condylar Rods with extendible pins (optional) - **AEB-NO**
1. Condylar Axis with Axis Pin, Centerpunched Calibrated
 2. Auditory Pin
 3. Condylar Track Assembly

Wear medical gloves when handling the device.

Clean, disinfect and sterilize the condylar rods and bite forks before first use and between uses.

Intended Use:

A dental instrument used to transfer the spatial relationship of the upper and lower jaws of the patient to the Articulator.

Indication for Use:

Instruments for reproduction of the teeth and jaw relation for analysis and extraoral fabrication of prostheses, other dental devices and in dental treatment planning.

User: Used by licensed dentist.

For additional information, see separate instructions for Dentatus Articulators.

NOTE:

For bite registration with Face Bow **AEB**, place the ends of the Calibrated Condylar Rods at the sites of the condyles of the patient.

After registration, the Calibrated Condylar Rods are attached to the Condylar Sphere Axis (1) on the Condylar Track Assembly (3).

With **AEB**, the rods should be locked on both sides with B (Set Screw) at a calibration of 6,5 during mounting on the Articulator.

For bite registration with Ear Bow **AEE**, place the Blue Ear Rods (I) in the external auditory meatus of the patient.

After registration, the Blue Ear Rods (I) are attached to the Auditory Pins (2) on the Condylar Track Assembly (3).

For this procedure it is IMPORTANT that the HCl is set to 40° and the Bennett-Angles to 20° to ensure the prescribed 12 mm distance from the Auditory Pins to the Condylar Axis.

With **AEE**, the Blue Rods should be locked on both sides with B (Set Screw) at a calibration of 7.2 during mounting on the Articulator.

Maintenance and cleaning:

After use, use alcohol wipes for disinfection. Calibration rods and Bite forks should be cleaned according to the symbols above.

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