Profin[®] PDX

Instructions — English	3
Instruktioner — Svenska	13
Anleitung — Deutsch	23
Instrucciones — Español	33
Instructions — Français	43
Istruzione — Italiano	53
Инструкции — български	63
Pokyny — Český	73
Instruktioner — Dansk	83
Οδηγίες — Ελληνικά	93
Ohjeet — Suomi	103
Instructies — Nederlands	113
Instruksjonene — Norsk	123
Instrukcje — Polski	133
Instruções — Português	143
Instrucțiuni — Română	153
Inštrukcie — Slovenský	163
คำแนะนำการใช้งาน — ไทย	173
Instrukcijos — Lietuvis	183

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Symbols used

Ultrasonic bath

Washer-disinfector for thermal disinfection

Sterilizable in a steam sterilizer (autoclave) at temperature specified

Do not re-use

Manufacturer

Reference/catalogue number

Batch code

Serial number

CE Mark

Consult instructions for use

Caution

Instructions — English

- Prior to initial use - always read the instructions for use.
- Only for US: prescription use – for use by dental clinicians only.

Indication for use

- Adjusting initial, finishing and polishing of dental surfaces and restorations.

Contraindication

- No known contraindications.

Precaution/Warning

Handpiece

Cross-contamination

- Clean, disinfect and sterilize the handpiece before first use and between uses.
- Follow the instructions in the instructions for use. The instructions are validated by the manufacturer.

Thermal

- Do not exceed 10,000 rpm.
- Verify the recommended rpm on the motor before start using the handpiece.
- Lubricate the handpiece between each use, debris can cause clogging and excessive friction.

Small objects

- Before start working: Check for secure seating by pulling the tip with your fingers. A firm resistance should be sensed.
- If tip comes out easily, try another tip or send handpiece for service.

Malfunction

- Do not disassemble any part of the handpiece.
- Lubricate the handpiece between each use, debris can cause clogging and excessive friction.

Vibration

- Discomfort for patient and dentist if speed is lower than 5,000 rpm.

Working tips

Cross-contamination

- Clean, disinfect and sterilize between uses.
- Do not re-use the EVA-tips. They cannot be cleaned properly.

Technical Description

Profin® PDX	
Description	Contra angle handpiece.
Motor coupling	E-fitting according to ISO 3964. Coupling dimensions: Long.
Material	Hard anodized aluminium.
Movement of working tip	Reciprocating 1,2 mm axial stroke.
Chuck	Collet.
Shank	Dentatus Snap-in.
Maximum motor speed	10.000 rpm.
Type of motor	Electrical (recommended) or air dental motor.
Recommended speed	Lamineer Tips: 5.000-10.000 rpm.
Torque	Depending on the torque of the motor.
Variation of tip movement: - Tip in a self-steering mode - Fixed positions	- For polishing procedure. - For creating anatomical corrections for esthetic alignments.
Accessories	
EVA-60	Plunger to eject tip.
PDX-F	Water spray clip, should external water be needed.

Table 1

Working tips to be used with Profin PDX	
Lamineer tips	Technical specifications and recommended use, see table 3.
EVA tips	Technical specifications and recommended use, see table 3.
Lamineer Tips IPR	Contact the manufacturer for separate instructions for use.
PER-IO-TOR	Contact the manufacturer for separate instructions for use.

Table 2

Profin® Lamineer tips - LTA

Cat#	Shank		Shape	Recommended use
	Colour			
LTA-150/2	Diamond	Red	LTA- 	Removal of gross overhangs and shaping of all restorative material - use tips with large particle size (150-75 µ).
LTA-S150/2				
LTA-100/2	Diamond	Green	LTA-S 	For shaping and polishing all prosthetic materials and enamel (75-50 µ).
LTA-S100/2				
LTA-75/2	Diamond	Gold	LTA-S 	For shaping and polishing all prosthetic materials and enamel (75-50 µ).
LTA-S75/2				
LTA-50/2	Diamond	Yellow	LTA-S 	Final polishing - use tips with small particle size (30-15 µ).
LTA-S50/2				
LTA-30/2	Dimond	White	LTA-S 	Final polishing - use tips with small particle size (30-15 µ).
LTA-S30/2				
LTA-15/2	Diamond	Violet	LTA-S 	Final polishing - use tips with small particle size (30-15 µ).
LTA-S15/2				
LTA-39/2	Tungsten Carbide	Black	LTA-S 	Tungsten Carbide does not scuff enamel. For final finishing of composite filling margins. (LTA-39 & LTA S36).
LTA-S36/2				
LTA-E50/2	Diamond	Yellow		For occlusal and concave surfaces.
LTA-C50/2	Diamond	Yellow		For finishing shoulders, bevels etc.
LTA-D150/2	Diamond	Red	LTA-D 	For shaping diverging surfaces and incisal edges.
LTA-D100/2	Diamond	Green		
LTA-D50/2	Diamond	Yellow		
LTA-D30/2	Diamond	White		
LTA-T50/2	Diamond	Yellow	LTA-T 	For opening incisal and gingival, approximal areas.
LTA-T30/2	Diamond	White		
EVA Tips for polishing with polishing paste (Single use plastic tips)				
EVA-2000/100	Plastic	Green		Universal tips, only used freely rotating.
EVA-5000/100	Plastic	Blue		

Table 3

How to use

Prepare your Profin for work

Apply the tip in the piston. Check for secure seating by pulling the tip with your finger. A firm resistance should be sensed. If tip comes out easily, try another tip or send the handpiece for service.


Mount the Profin handpiece on your dental motor.

Internal water or spray should be turned off on the unit. Spray is under normal conditions not necessary for cooling.

External water connection can be used with water spray clip. This is optional.


The tip can be used in two different positions:

- In position A, the tip is free to rotate to follow contours.
 - In position B, the tip is fixed in a position of convenience.
- To change a fixed position, pull out the tip slightly, turn and push it back into a new position.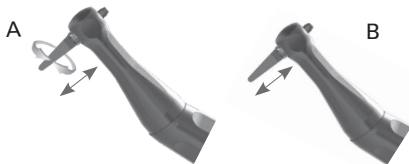

How to eject the tip

Use the EVA-60 plunger to safely eject the tip.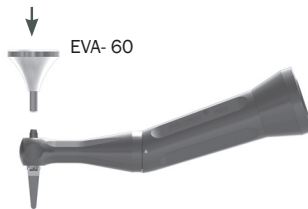

Selection of tips

The selection of diamond grit is depending on the amount of material that has to be reduced. A general recommendation is the more material the more coarse diamonds. The removal of material and finishing should be made in steps.

NOTE: Never go from super-coarse to super-fine in one step, but use intermediate medium-coarse tips.

NOTE: The black Tungsten-Carbide-coated tips can always be used for finishing when there is a risk for abrasion on enamel, since Tungsten Carbide does not scar or scuff enamel.

NOTE: The wearing of the Diamond Tips is low. The durability is depending on the circumstances, for continued efficiency it is recommended not more than 30-40 times.

Examples of use:

- On enamel, metallic, ceramic and composite restorations.
- For refining gingival and incisal embrasures to natural forms.
- To contour and individualize anterior restorations.
- To reshape and polish tooth contacting areas.
- To chairside correction and refinishing margins prior to cementation.

Reduction of overhangs and excess cements

Apply the tip on the material to be reduced or finished in parallel to the tooth contour (1).


Start the motor, move the handpiece slowly up and down, or in and out in order to remove debris (2).

Change for a finer diamond when the bulk of material has been reduced. Finish with 30 or 15 micron tip and polish with the plastic tips together with a polishing paste.

When cementing laminates on incisors, the excess material in the palatal concavity can easily and safely be removed with a bent tip (3).

Adjustment of inlay margins

The margins can normally be finished together with removal of excess luting material (4). The procedure is similar to removal of overhangs. If inlays/onlays are being made on adjacent teeth, finish each tooth after cementation for easier access, and polish when all have been cemented.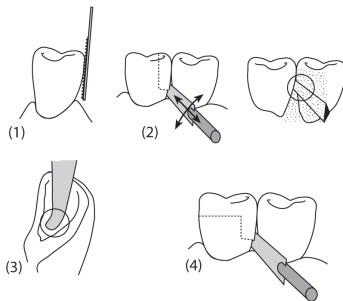


Removal of subgingival excess cements

The tips are non-traumatic to soft tissue and can be used subgingival (5). Select a suitable tip (normally this procedure does not require super-coarse or coarse tips). Apply the tip on the subgingival surface and start the motor. The tips can be either fixed or in a swirl position whichever provides the best tactile sense. For subgingival polishing the blue spatula-shaped tip EVA -5000 may be used.

Additional uses for Profin:

- Shaping and adjustment of corners (6)
- Shaping of approximal embrasures (7)
- Shaping of contours (8)
- Shaping/adjustment of incisal edges (9)
- Adjustment of occlusion (10)
- Finishing of shoulders (11)
- Preparation of bevels (12)

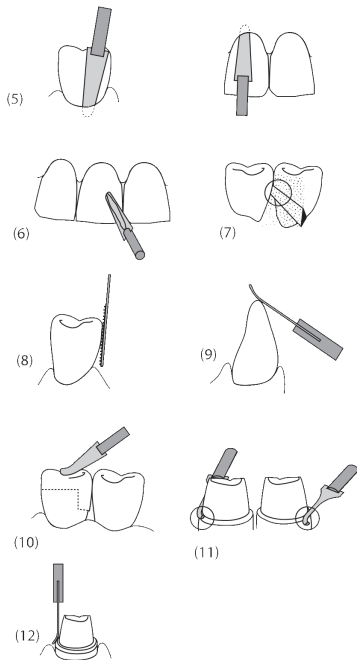Processing Instructions

Profin PDX	
WARNINGS	<p>Do not immerse the handpiece into disinfection solutions or into ultrasonic bath. The solutions may contain chemicals that may impede or freeze the handpiece drive mechanism.</p> <p>Do not use washer disinfectant. Anodized aluminium is not suitable for washer disinfectant. The hand piece is not tested for repeated cycles in washer disinfectant.</p> <p>The handpiece shall NOT be disassembled.</p>
Limitations on processing	<p>Sterilize the handpiece for a maximum of 250 cycles. We recommend that you send the handpiece to the manufacturer for service after 250 uses/cycles, or every 12 months (whichever comes first).</p>
INSTRUCTIONS	
Initial treatment at the point of use	<p><u>Instruction</u></p> <ol style="list-style-type: none"> 1. Flush instrument on unit before removal from tubing. 2. Thereafter remove the tip.
Preparation before cleaning	<p><u>Instruction</u></p> <p>Ensure the tip is removed from the handpiece.</p> <p>If heavily soiled, clean first with soaked cloth or pre-soaked wipes.</p> <p>The maximum period of time between start of use and start of cleaning process shall not exceed 2 hours. ²⁾</p>
Cleaning: Automated ²⁾	<p><u>Equipment</u></p> <p>Automated machine for dental handpieces.</p> <p><u>Detergent</u></p> <p>In accordance with manufacturer's specification for the use of the automated machine.</p> <p><u>Instructions</u></p> <p>In accordance with the manufacturer's specification for the use of the automated machine.</p> <p><i>If soil is visible after cleaning with automated machine, perform manual cleaning.</i></p>
Cleaning: Manual	<p>External</p> <p><u>Equipment</u></p> <p>Non-abrasive, medium hard brush.</p> <p><u>Detergent</u></p> <p>Low alkaline.</p> <p><u>Water quality</u></p> <p>Deionized (DI) <40 °C or Reverse osmosis (RO) <40 °C.</p> <p><u>Instruction</u></p> <ol style="list-style-type: none"> 1. Brush off under non-stagnant water. 2. Rinse under non-stagnant water for 15 seconds.

	<p>Internal</p> <p><u>Equipment</u> Cleaning spray/dry spray intended for dental handpieces.</p> <p><u>Instruction</u> In accordance with manufacturer's specification for the use of the spray(s). <i>Hold check against clean, white wipe and repeat procedure until no visible soil remains.</i></p>
Disinfection: Automated	Not applicable. See WARNINGS.
Disinfection: Manual	<p><i>Caution:</i> The handpiece must be cleaned both internally and externally before disinfection.</p> <p>External</p> <p><u>Equipment/Detergent</u> Soaked cloth or pre-soaked wipes/Disinfectants suitable for anodized aluminium.</p> <p><u>Instruction</u> In accordance with manufacturer's specification for the use of the disinfectants.</p> <p>Internal</p> <p><u>Equipment</u> Disinfection spray intended for dental handpieces.</p> <p><u>Instruction</u> In accordance with manufacturer's specification for the use of the disinfection spray.</p>
Lubrication: Automated	<p><u>Equipment</u> Automated machine for dental handpieces.</p> <p><u>Lubrication</u> In accordance with manufacturer's specification for the use of the automated machine.</p> <p><u>Instruction</u> In accordance with manufacturer's specification for the use of the automated machine.</p>
Lubrication: Manual	<p><u>Process chemicals</u> Lubricants intended for dental handpieces.</p> <p><u>Instruction</u></p> <ol style="list-style-type: none"> 1. Insert the spray can nozzle into the motor input of the handpiece. 2. Push the button of the can until the liquid coming out of the head of the handpiece is clear. 3. Wipe off excess fluid with a paper towel.

Inspection and Testing	<p><u>Inspection</u> Visually inspect for damage and wear.</p> <ul style="list-style-type: none"> - No scratches. - Check that there is no gap between head and body. - Check that screws are even with surface. <p><u>Testing</u> Check the chuck by inserting a tip. You should not be able to easily pull out the tip with your finger. If any of the above appears, do not use the handpiece and contact the manufacturer.</p>
Packaging for sterilization	<p><u>Equipment</u> Sterile goods packing, ³⁾</p> <ol style="list-style-type: none"> 1. Pack individually. 2. Check that the bag is not stretched. 3. Check correct sealing.
Sterilization 	<p><u>Caution:</u> The handpiece must be cleaned and disinfected before sterilization.</p> <p><u>Equipment</u> Saturated steam sterilizer including fractionated pre-vacuum before sterilization phase and final post-vacuum drying (B-process). ⁴⁾</p> <p><u>Temperature</u> 134 °C (273 °F)</p> <p><u>Time</u> Minimum of 3 minutes.</p> <p><u>Pressure</u> 3.06 bar (27 psi).</p> <p><u>Vacuum drying</u> Minimum 3 minutes.</p>
Storage	<p>Protect from dust, dirt and humidity, UV-light (sunlight) and extreme temperature (above room temperature). Before use ensure the sterile bag is dry, whole and clean.</p>
Additional information	<p>¹⁾ If delayed cleaning, corrosion may affect the surface. ²⁾ Automated machine for both internal and external cleaning. ³⁾ Sterile goods packing according to EN 868-5. ⁴⁾ Steam sterilizer shall be in accordance with standards EN 13060, EN 285 and EN ISO 17665-1. All equipment, detergents, spray, lubricants and disinfections solutions shall be CE-marked as per directive 93/42/EEC.</p>

The instructions provided have been validated by the manufacturer of the medical device as being capable of preparing a medical device for reuse. It remains the responsibility of the processor to ensure that the processing, as actually performed using equipment and materials in the processing facility, achieves the desired result. This requires verification and/or validation and routine monitoring of the process.

Processing Instructions

Lamineer Tips	
WARNINGS	Use chemicals suitable for diamond coated stainless steel instrument.
Limitations on processing	See inspection.
INSTRUCTIONS	
Initial treatment at the point of use	Soak in solution. If debris remains use a cleaning stone suitable for diamond instruments.
Cleaning: Automated 	Washer disinfecter (in accordance with standard ISO 15883). Without or with mild detergents.
Cleaning: Manual 	Ultrasonic bath. Detergent. In accordance with manufacturer's specification for the use of detergent.
Disinfection: Automated 	Washer disinfecter (in accordance with standard ISO 15883). Without or with mild detergents.
Disinfection: Manual	Disinfection bath. Disinfectant. In accordance with manufacturer's specification for the use of disinfectant. Thorough rinsing.
Inspection	Visually inspection for appearance.
Packaging for sterilization	Sterile goods packing according to EN 868-5.
Sterilization 	<u>Equipment</u> B-type steam (in accordance with standards EN 13060, EN 285 and EN ISO 17665-1). <u>Temperature/Time/Pressure</u> 134 °C (F 273 °F)/Minimum of 3 minutes/3.06 bar (27 psi). <u>Vacuum drying</u> Minimum 3 minutes.
Storage	Protect from dust, dirt and humidity, UV-light (sunlight) and extreme temperature (above room temperature).

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Dentatus AB